

**Instructions for service on INVACARE rollators
P452E/2 Spatio and P452E/3 Banjo**

Refurbishment service

The following check-ups and controls are to be done when maintaining above mentioned rollators

GENERAL

	Examine that/if:	To do:	Comments:	OK
1	all screw joints are properly tightened.	Retighten all loose screws using glue Loxeal 5503 were appropriate.	If there are any doubts regarding the appropriate function of the rollator, please contact Dolomite.	

WHEELS

	Examine that/if:	To do:	Comments:	OK
1	the wheel bearing not is damaged.	The wheel must spin easily. If not, the wheel shall be exchanged.	When changing wheel, use a new locking nut!	
2	the tread is not damaged.	Change the wheel.	Use new locking nut!	
3	the wheel bolts are properly tightened.	Tighten the bolts, (use glue Loxeal 5503 if locking nut not is available).	Use new locking nut!	
4	the front fork is properly assembled into the frame.	Tighten the front fork in the frame with 60 Nm.		
5	the front fork has no damages.	Change the fork.		

BRAKES

	Examine that/if:	To do:	Comments:	OK
1	the brake flap is not worn out.	Exchange the brake flap.		
2	the brake flap returns after braking.	Change the brake flap or wire and cover.		
3	the brake wire moves easily in the wire cover.	Change wire and, if needed, cover.		
4	full braking action is obtained.	Adjust the braking action with the wire adjusting devices.		
5	end parts on wire covers are not damaged.	Change the wire cover.		
6	brake handle/housing is not damaged.	Change the brake handle.		

FOLDING MECHANISM

	Examine that/if:	To do:	Comments:	OK
1	the walker locks and remains in unfolded position.	Change folding lock mechanism.		

HANDGRIPS

	Examine that/if:	To do:	Comments:	OK
1	the handgrips not are damaged.	Change grip / grips		
2	the hand grips can not be easily rotated.	Change grip. Clean the tube with a grease solvent that does not contain oil, an example of suitable liquid is windscreen cleaning fluid.		

HANDLE FIXATION

	Examine that/if:	To do:	Comments:	OK
1	the locking levers are locking the handle tube properly.	Change the locking levers.		

FRAME

	Examine that/if:	To do:	Comments:	OK
1	there are no mechanical damages on the frame.	Scrap the rollator.		
2	the handle tubes are not bended.	Scrap the rollator.		

SEAT

	Examine that/if:	To do:	Comments:	OK
1	the seat is not damaged.	Change the seat.		

Service instructions

INVACARE ROLLATORS



**CONTROL OF ROLLATORS AS ABOVE ROUTINES SHALL BE CARRIED OUT REGULARLY.
THE FREQUENCY SHALL BE RELATED TO THE INTENSITY OF USE AND WHAT STRESS
THE ROLLATOR HAS BEEN EXPOSED TO.**

Rollator model: _____
Serial number: _____

<i>Service made by:</i>
Name: _____
Date: _____
Sign: _____

DOLOMITE AB - V. Götgatan 5, 334 32 ANDERSTORP - Tel: 0371 58 84 00 Fax: 0371 170 90